NOVAVAC PVT

PNEUMATIC VACCINATOR

USER'S MANUAL

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NOVAVAC PVT

Table of Contents

- 1. Safety and Hygiene Instructions..... 1
- 2. Introduction and Installation of the Device..... 3
- 3. Injectors..... 9
- 4. Cleaning and Maintenance..... 16
- 5. Periodic Replacement of Parts..... 21
- 6. Technical Points..... 21
- 7. Possible Problems and Alternative Solutions..... 22
- 8. List of Device's Components and Spare Parts..... 24

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1. Safety and Hygiene Instructions

- Please read this instructions manual before you operate the vaccinator.
- Apply recommended values and methods for operating the device and vaccination processes.
- In case the operator is injured during vaccination process immediately call a doctor and get medical help.
- Keep compressed air disconnected while replacing a needle or doing a maintenance.
- Before making any changes on needle depth and treatment block the device has to be brought to needle adjustment position at which vaccination is not applied. Perform this when the air is on by using the knob for needle protrusion (#8 in the picture below).
- The device should be cleaned using compressed air, dampened cloth and alcohol. The person who does the cleaning must know how to use the device and follow the instructions.
- For replacement of defective parts you should contact Novakim Mectech and use only original parts.
- If you need to carry the device always do it using your both hands and keep the device in horizontal position. Do not carry the device with one hand and in vertical position.
- You should evaluate the vaccination quality by vaccinating 15-20 chicks before you begin actual vaccination process.
- Device should receive regular cleaning and maintenance.

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WARNING

- Be vigilant and precautionous against the accidents which may happen during vaccination.
- Make sure you read and understand the instructions manual before you operate the device.
- Before you make any action on the device (maintenance, service, cleaning, etc.) disconnect the air supply.
- Consider to have the operator vaccinated against tetanus as a precaution against operating accidents.

IN CASE OF THE ACCIDENTS DURING VACCINATION

- Remove any rings or ornaments from your hand as a precaution against possible swelling.
- Clear the injection site off the vaccine by making it bleed, and clean using copious amount of water and alcohol.
- IMMEDIATELY go to the nearest hospital and take the injected vaccine's bottle and package insert with you.

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
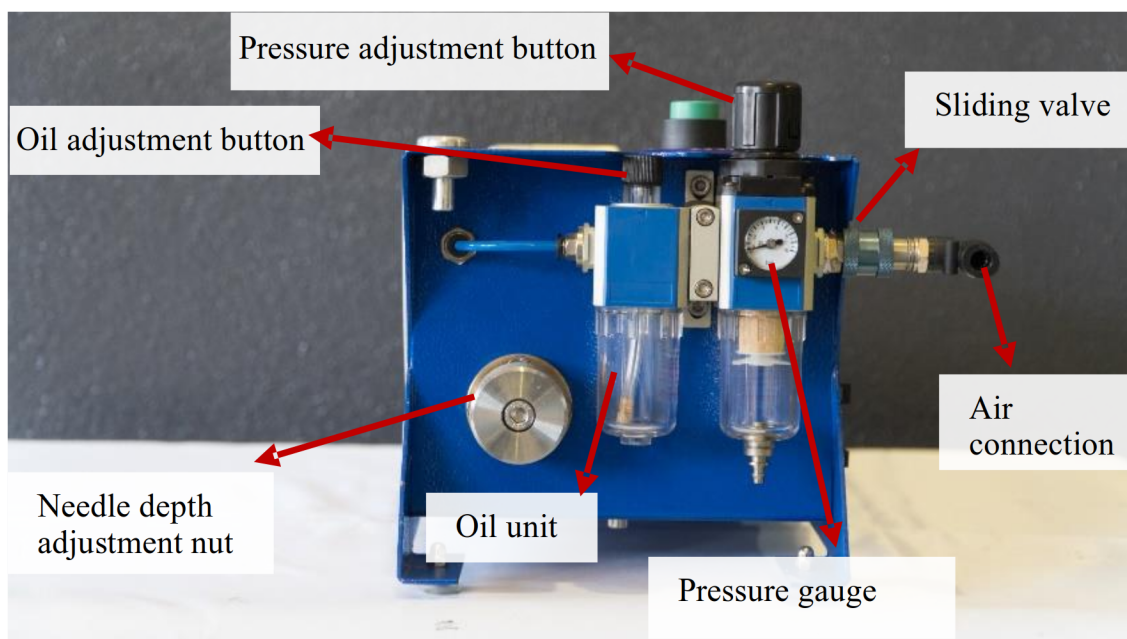
2. Introduction and Installation of the Device

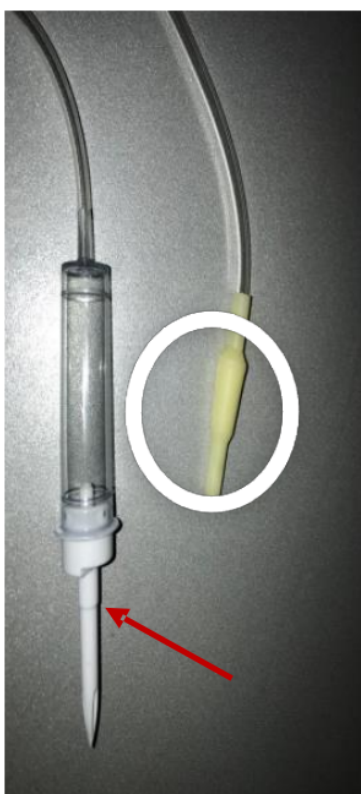2.1. Connecting the vaccine to the device

- Insert the plastic spike of a sterile transfusion set (shown with red arrow in the image) in the vaccine's transfer cap.
- The part which is shown in a circle in the image is connected to the vaccine inlet valve of the injector. This part of a sterile transfusion set comes with a needle attached. Carefully remove that needle and place in a safe place.
- Adjust the flow control regulator of the set to open position.
- Hang the vaccine bottle on the device's hanger and let the vaccine flow until there is no air in the tubing.
- When the tubing is filled with vaccine connect the end to the injector which you will use.
- In order to make the vaccine flow into the injector you'll have to create a suction by pressing at the end of the injector a few times. Repeat this until you see the vaccine dispensing from the injector.

- Once you completed this, the injector is ready to be operated.

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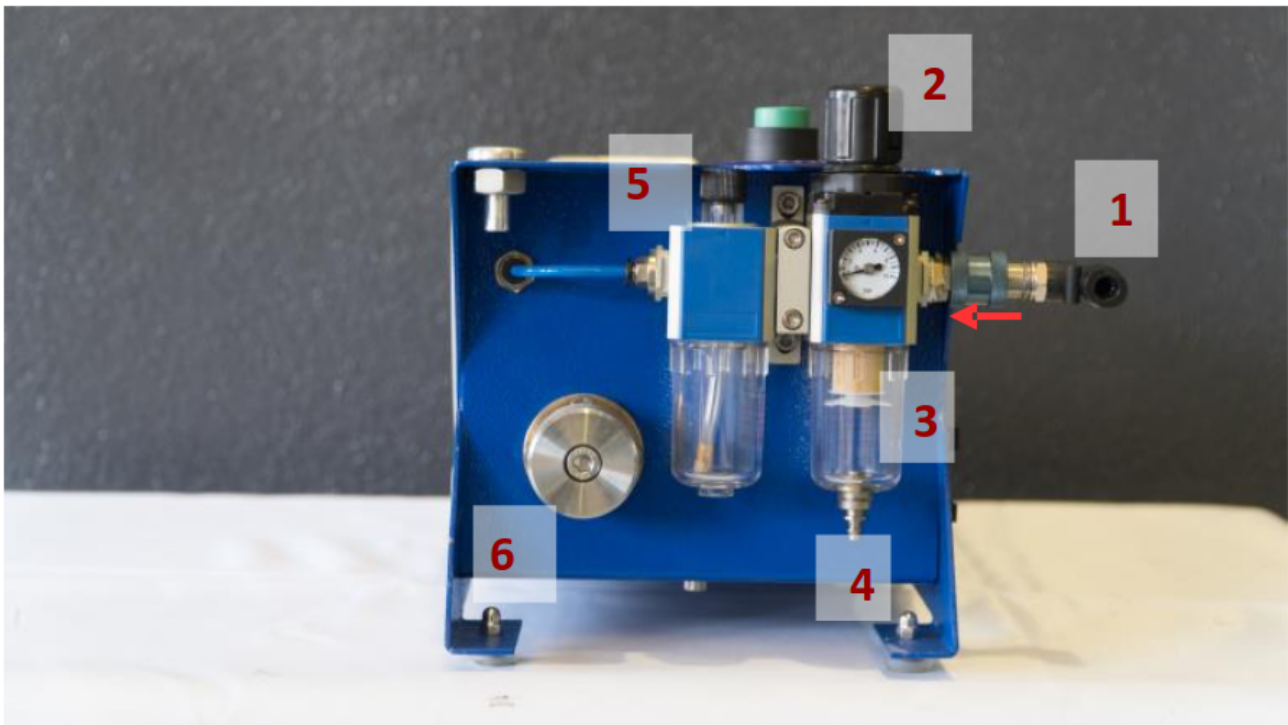
2.2. Air Connection and Pressure Conditioner

- Device needs compressed air connection to work.
- Connect the air tubing from the compressor to the air inlet (#1) of the device.
- Make sure there is no air leakage.
- There is a sliding valve at the connection site which controls air flow. When the valve is moved towards the device (to the direction of green arrow) it is opened. When the valve is moved in the opposite direction air supply is closed and air in the device is released.
- After the connection is made the pressure has to be adjusted to 4-6 bar.
- Lift the pressure adjustment knob (#2) to adjustment position in order to adjust the pressure.
- You can increase or decrease the air pressure by turning the knob in +/- directions.
- You can follow the pressure adjustment on the gauge.

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- When the pressure gauge shows 4-6 bars the air required for the device to work will be met.
- After you adjusted the pressure press the knob down to its original position.
- Pressure conditioner's filter unit (#3) filters dust and water which may cause damage to the device.
- When this container begins to fill, it should be removed by turning and cleaned.
- If water accumulates in the container, it can be drained by pressing the stopcock under (#4).
- On the other side of the pressure conditioner there is an oil unit which helps the pneumatic parts to function effectively. Oil amount can be adjusted using the oil adjustment knob (#5) on it.
- Oil amount in the container will decrease depending on the frequency of operation. It is recommended to make additions before the oil is totally run out.
- Only oil suitable for pneumatic operations should be added. You may ask assistance of authorized technical service for oil supply.

2.3. Application Counter and Its Features

- The counter's main feature is that it can be set to a number from which it counts down, and when it reaches **000** the device stops vaccination.
- You should press the reset button in order to go the set number and restart operation.
- The counter counts down from the set number.
- It is very simple and easy to use.

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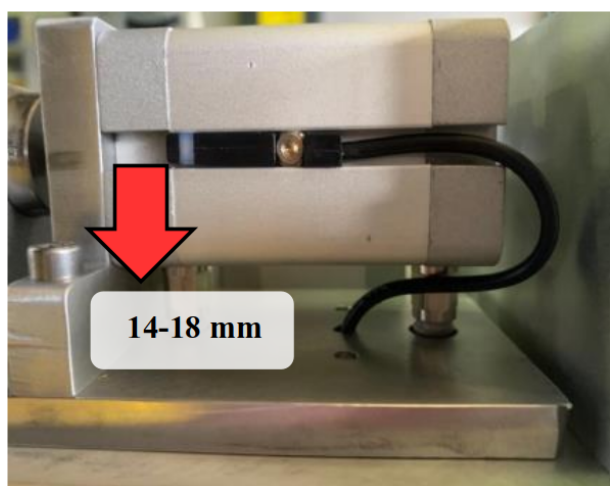
- With each press on the button below each digit changes the respective digit and so the target number can be set.
- In case of an unexpected or contrary situation please contact technical service.

2.4. Total Counter and Its Features

- Total counter doesn't require an external feed.
- It has a long-life and durable battery.
- Each number indicates one vaccination.
- By means of the counter sensor on the piston, it counts the number of vaccinations and digitally displays it.
- If the counter ever stops counting, the sensor is the part to be checked first. It should be positioned 14-18 mm apart from the front side of the piston.
- The Reset button on the counter resets it.
- In case of an unexpected or contrary situation please contact technical service.

2.5. RESET Button

- The RESET button is used to reset the counter and continue vaccination when the set number is reached and operation is stopped.
- Before beginning vaccination pressing this button once will be enough for the device to work systematically.

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2.6. Positioning Switches

- When the positioning switches' arrows are directed up they are closed.
- Water injector
- When active, the oily injector is used for double vaccinations. Two injectors work at the same time.
- The other positioning switch (#8) is used to adjust needle's depth and vaccination position.
- If you turn on this switch when device's cover is closed you will see the needle protrude out of the cover to vaccination position.
- When you are using this switch make sure you keep your hand or the chick away from the needle's way.
- After you adjusted needle depth and position turn the switch to off position and keep it so during vaccination.

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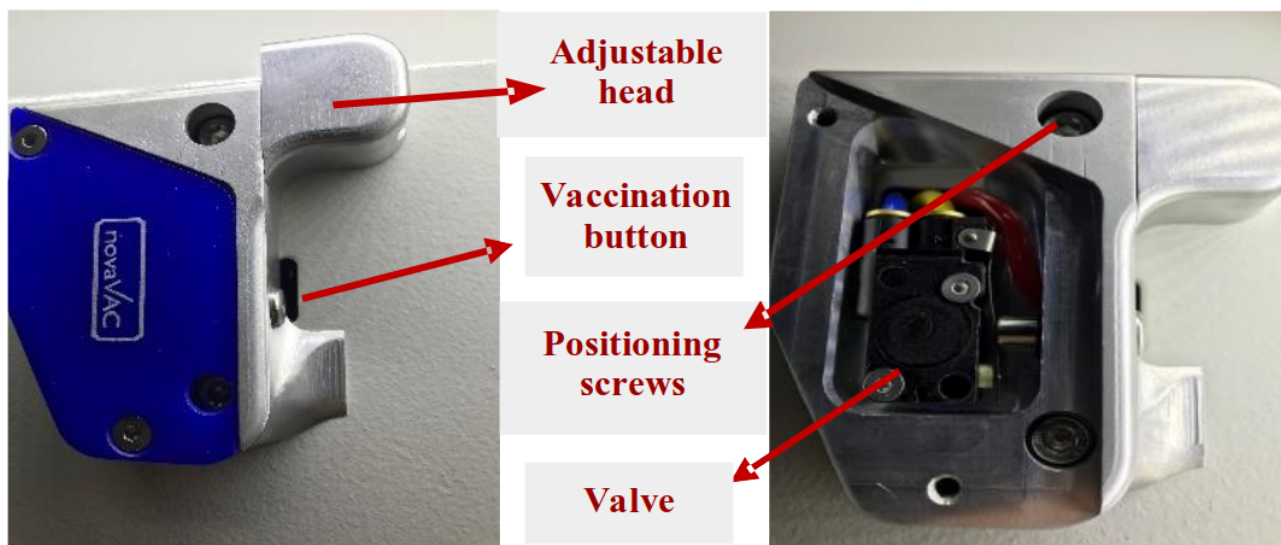
2.7. Treatment Block and Its Adjustment


- Positioning screws are used to position the treatment block.
- Position of the treatment block is the most important factor to optimally position the chick in order to make correct vaccination.
- By loosening the positioning screws the block can be re-positioned down/up and right/left.
- When adjusting the position, the needle must be in adjustment position by using positioning switch #8.
- Adjustable block can be moved down or up to let the chick's head and neck be positioned in the most convenient way.
- This adjustment is used to minimize vaccination errors with heavier chicks and to fit user's custom.
- Vaccination button and the valve behind it is used to start vaccination. When the chick is positioned its neck is pressed against the button and vaccination is done.
- Each press on the button performs one vaccination. Each chick must be vaccinated only once.
- When the device's cover is closed you should never press the vaccination button except for vaccination purpose. Otherwise, you can inject the vaccine in your hand or you can waste the vaccine.
- You should clean the vaccination button at the end of each working day in order to protect it from poultry dust and vaccine residues.

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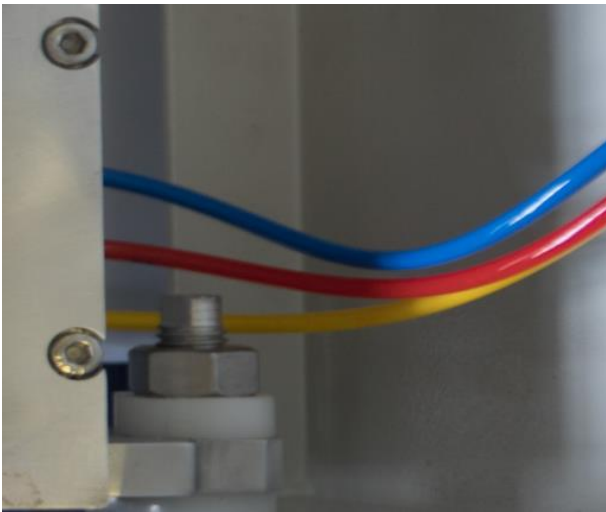
2.8. Air Tubing

- At the front side of the device, under the cover there are three air tubing going to the treatment block.
- The blue tubing feeds the valve of treatment block.
- Yellow tubing carries the air which returns from the treatment block and starts vaccination.
- Red tubing carries the air to the treatment block for warning and cleaning after the device reaches the set number and stops.
- In case of a damage or displacement of the tubing you should check the situation and call technical service.

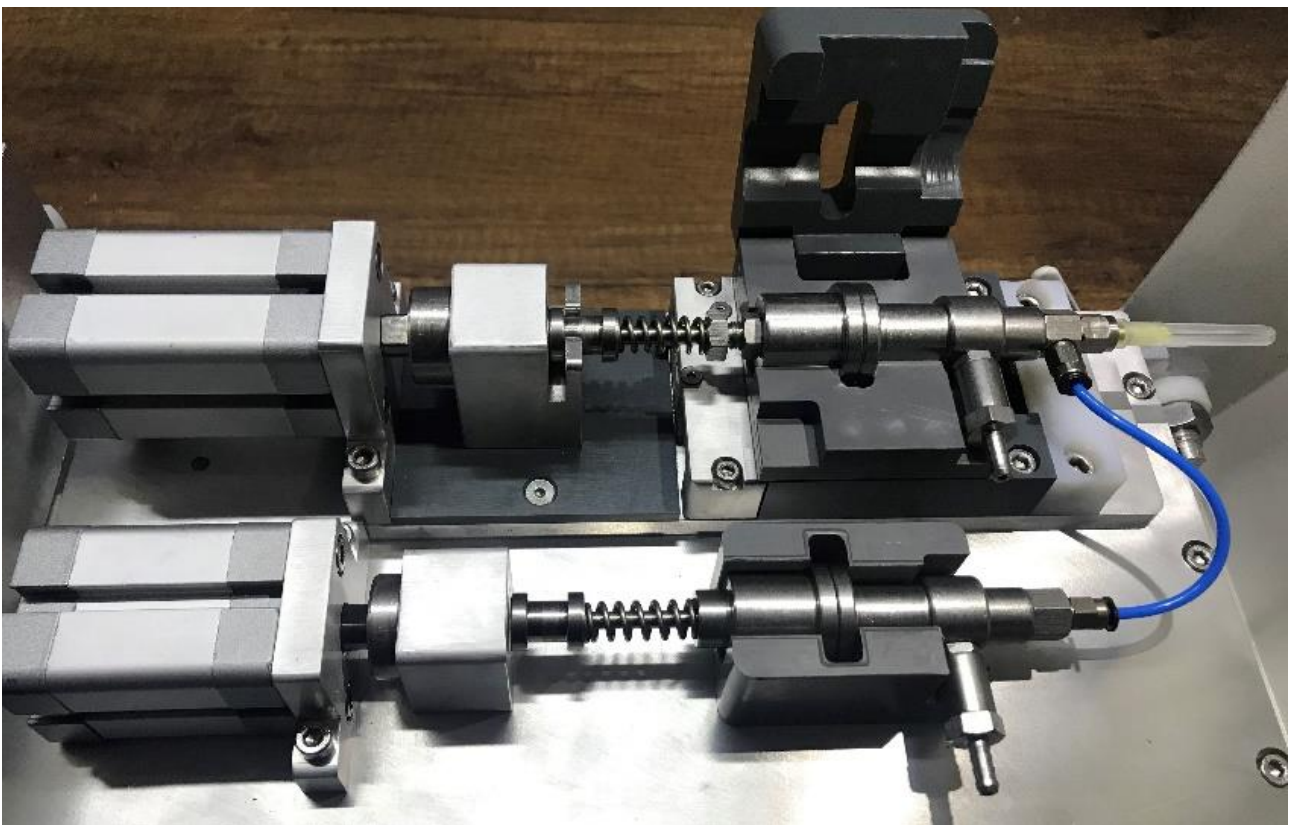3. Injectors


- The device has two injectors, one fixed and one adjustable.

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- The adjustable injector carries the needle and performs vaccination.
- A feature of the adjustable injector is that its dosage can be adjusted between 0.05 ml and 0.3 ml.
- Dosage can be adjusted using the dosage adjustment screw by loosening the dosage adjustment nut at the back of the injector.
- Fixed injector is calibrated and set to deliver 0.1 ml.
- This injector is used as secondary injector during double vaccinations and it delivers vaccine via adjustable injector with the help of its connection tubing.
- Fixed injector should not be used alone.

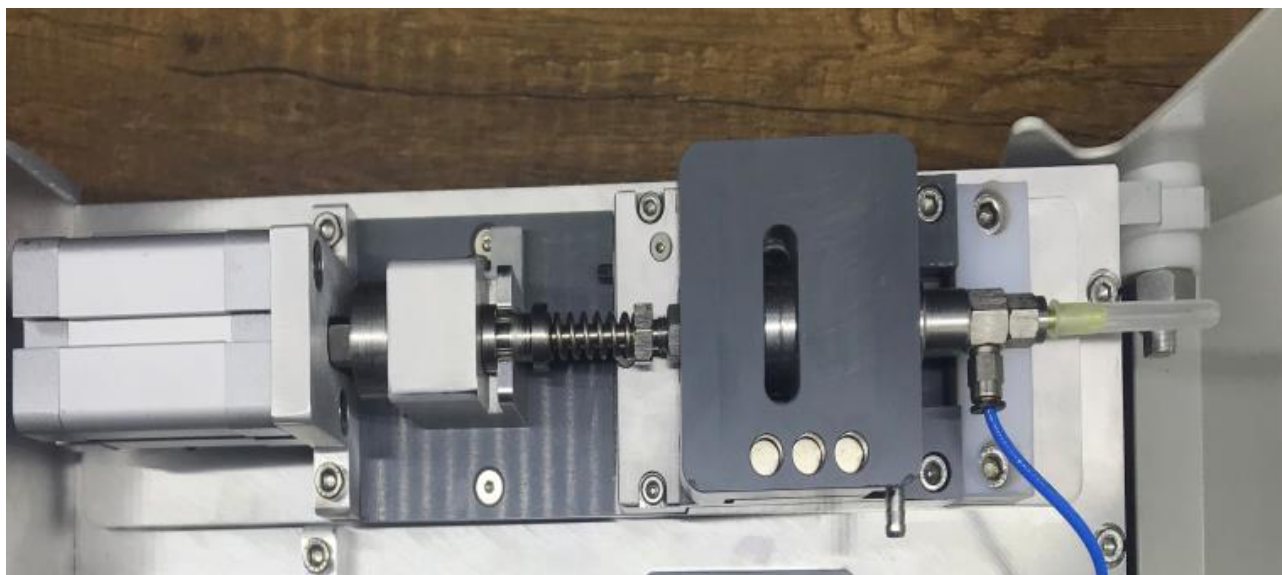3.1. Adjustable Injector

- Basically it is composed of two check valves and an injection system.
- It easily takes the vaccine and applies exact dose because of its advanced check valve system.
- Dosage can be adjusted using the dosage adjustment screw by loosening the dosage adjustment nut at the back of the injector.

3.2. Fixed Dose Injector

- It works in a similar way with adjustable injector.

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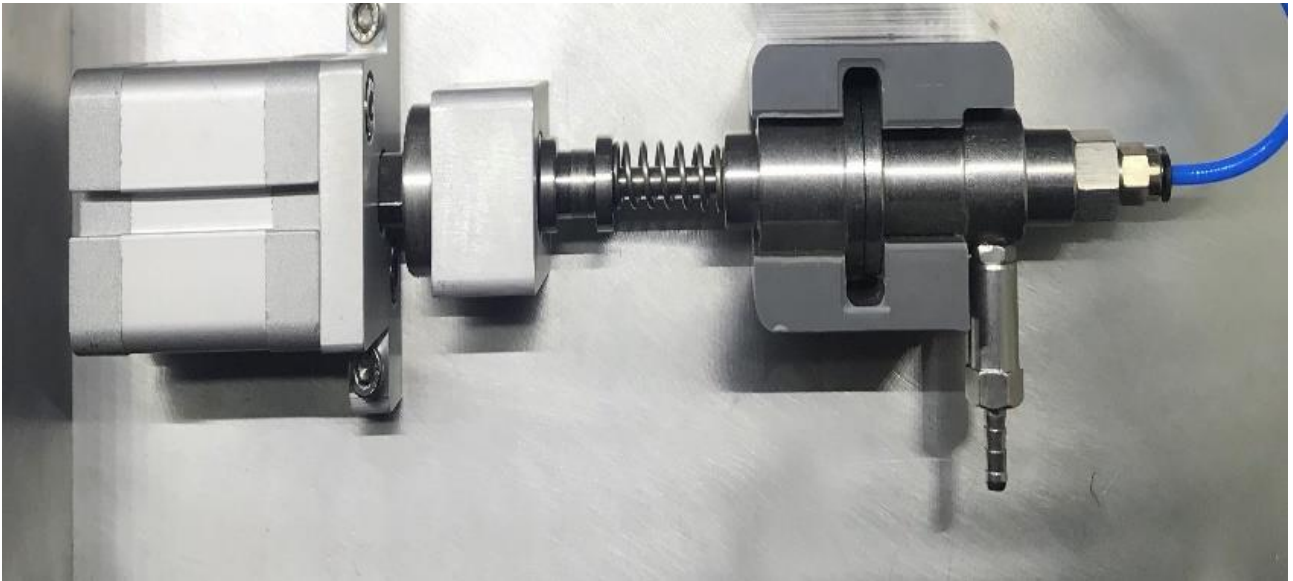
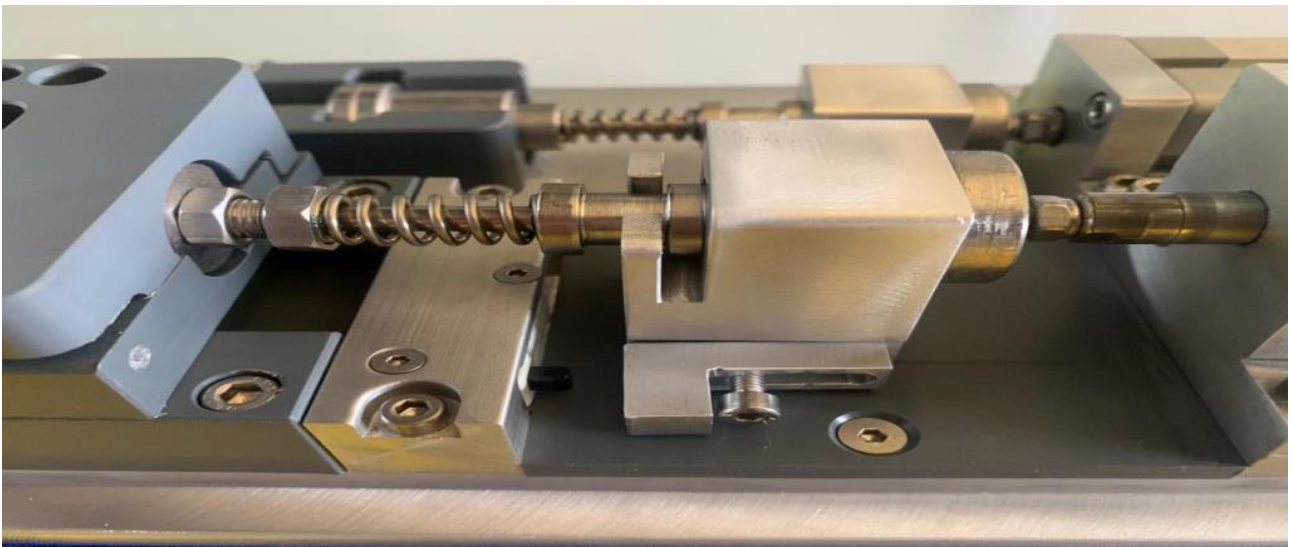
- Dosage is set to 0.1 ml.
- It is connected to adjustable injector with a tubing.
- As it makes the vaccination through adjustable injector, it should not be used alone.

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- It is activated with the help of the valve at the back of the adjustable injector.
- The L shaped part on the block on which the back of the injector is mounted activates the fixed injector by touching the valve each time the needle protrudes.
- In order to position this L shaped part move the needle forward using the position switch #8, and then when the part is in position pressing the black button of the valve tighten it using the fixing screw.
- In case of malfunction or failure first make sure the position switch #7 is open and then check the L shaped part.
- If the fixed injector still can not be activated, contact technical service.

3.3. Needle Installation

- Install the needle using the needle nut in front of the adjustable injector.

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- Without removing the protector cover of the needle, unscrew the nut and place the needle through the hole of the nut.
- Place and lightly screw the assembly on the injector and adjust the direction of needle tip.
- Direction of the needle should be adjusted to be facing 45° to the right and up when you are facing the device from its front.
- After you adjusted the direction of the needle tighten the nut. Make sure there is no gap and the needle is fixed in position.
- To remove the needle, first unscrew the nut and remove the needle through its hole.
- Use the adjustment nut #6 to adjust the needle depth. You can increase or decrease the needle's protrusion from the device by turning this nut to the right or left.

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